


Patient Directions for Use

Before Use Make Sure Medicine Is Ready When You Need It


Look at the medicine in the Adrenaclick regularly. If it looks cloudy, is discolored, has particles in it, or if the expiration date has passed, the Adrenaclick should be replaced.

Do NOT remove **GRAY** caps until you are ready to use the Adrenaclick.

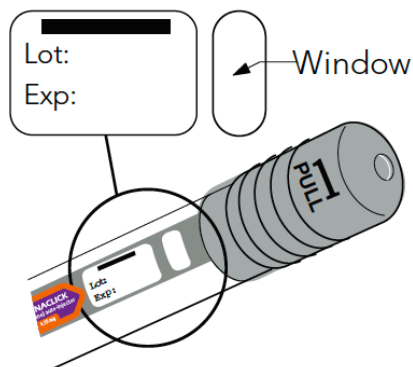

STEP B

- Put the **RED** tip against the middle of the outer side of your thigh (upper leg) as shown. It can go through clothes.
- Press down hard until the needle enters your thigh (upper leg) through your skin. Hold it in place while slowly counting to 10.
- Remove the Adrenaclick from your thigh.
- Check the **RED** tip. If the needle is exposed you received the dose. If needle is not visible, repeat Step B.

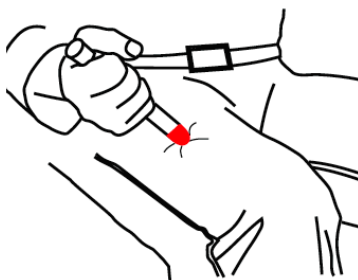


Get emergency medical help right away.

Call 911

The following instructions should be followed for both Adrenaclick 0.15mg and Adrenaclick 0.3mg. Additional video instructions on the use of Adrenaclick are available at www.adrenaclick.com.

STEP A

- Pull off **GRAY** end cap with the [1]; you will now see a **RED** tip. Never put thumb, finger, or hand over the **RED** tip.
- Pull off **GRAY** end cap with [2].

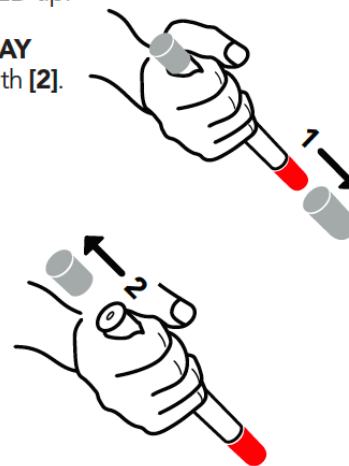

AFTER USE/DISPOSAL

- The remaining liquid that is left after this fixed dose cannot be further administered and should be discarded.
- Put the auto-injector, needle first, into the carrying case.
- Put the light gray case top over the non-needle end of the Adrenaclick unit and close the case.
- Give your used Adrenaclick to a healthcare worker for proper disposal. Do NOT throw away in a regular trash can.

